

Reprocessing the Ardo breastpumps for rental centers

Carum, Ardo Bellis and Calypso



To safeguard the hygiene of the Ardo breastpumps Ardo Bellis, Carum and Calypso, Ardo medical AG recommends that the surface of the breastpumps and the Carum packaging box be reprocessed immediately after return. The pump surface is reprocessed using a non-phenol disinfectant.

Abrasive agents should not be used. If pumpsets or other accessories have been returned, they must be disposed of because they are intended for personal use.

Important notes

Every time the device is cleaned, it must be switched off, and the power cable must be disconnected from the power supply. Failure to do so may result in an electric shock!

If the pump is contaminated by overflow or another cause, the contaminated parts must be disposed of by the user according to local guidelines.

Do not use cleaners or disinfectants based on phenol! Failure to follow this instruction may destroy the plastic materials due to the formation of fine cracks.

Reprocessing

The device must be cleaned and disinfected each time before it is supplied to a customer. This also applies for the first use after delivery because none of the Ardo breastpumps are supplied disinfected.

Please also comply with the legal provisions and hygiene regulations that apply in your country.

Reprocessing the Ardo breastpumps for rental centers

Work station

Unpack and reprocess the Ardo breastpumps at a work station that can be disinfected by wiping down. Keep processed Ardo breastpumps in a separate location to prevent contact with pumps that have not been cleaned and disinfected.

Preparation

- Prepare a ready-to-use disinfectant that is compatible with the material (see below). Read the product information on the product and check the expiry date before use.
- For your own protection, we recommend that you wear gloves. Please note the safety data sheet for the disinfectant used.
- Open the packaging and take out the Ardo breastpump. Dispose of the accessories according to country-specific guidelines.

Operation

Disconnect the Ardo breastpump from the power supply.

1. Before disinfecting the breastpump, remove any visible soiling from the surface of the Ardo breastpump and its power cord using a damp cloth and a commercially available mild detergent.
2. Do not allow cleaning fluid to penetrate the connection openings for the pumpset or the electrical connections.
3. Do not clean the pump under running water or in a water bath.
4. After cleaning, dry the surfaces with a clean, dry cloth.
5. Reprocess the surface of the breastpump using a disinfectant that is compatible with the materials (see below). Allow the disinfectant to take effect as described in the manufacturer's instructions.
6. Disinfect your work surface after reprocessing the Ardo breastpumps.

Reprocessing the Ardo Bellis breastpump



Additional information for Ardo Bellis

The Memory function must be reset before the next **leasing**.

Possible cleaners

Product name	Manufacturer / supplier	Ingredients
Surface disinfection / aldehyde-free		
Biguanid Fläche N	Schumacher / Dengg	Quaternary compound
Biguanid Fläche NR	Schumacher / Dengg	Quaternary compound
Surface disinfection / oxidising agent		
Perform	Schülke+	Peroxide compound
Surface disinfection / alcohol-free		
mikrozid® sensitive liquid	Schülke+	Biocide

Reprocessing the Ardo Carum breastpump



Additional information for the Carum container

The Carum container can be cleaned with a conventional detergent.

Possible cleaners

Product name	Manufacturer / supplier	Ingredients
Surface disinfection / aldehyde-free		
Biguanid Fläche	Schumacher / Dengg	Quaternary compound
Incidin Plus	Ecolab	Alkylamine
TPH protect	Schülke+	Quaternary compound
Surface disinfection / aldehyde-containing		
Antiseptica Combi Surface	Antiseptics	Quaternary compound / aldehyde
Surface disinfection / ready-to-use / alcohol-resistant surfaces		
Incidin liquid	Ecolab	Alcohols
Mikrozyd AF liquid	Schülke+	Alcohols
Surface disinfection / ready-to-use / non-alcohol-resistant surfaces		
Acryl Des	Schülke+	Quaternary compound
Bacillol 30 Foam	Hartmann / Bode	Alcohols / alkylamine derivative
Surface disinfection / disinfectant wipes		
Kohrsolin extra tissues	Hartmann / Bode	Aldehyde / quaternary compound
Mikrobac Tissues	Hartmann / Bode	Quaternary compound
Sanicloth active wipes	Ecolab	Quaternary compound
Surface disinfection / oxidising agent		
Descogen liquid	Antiseptics	Peroxide compound
Incidin Active	Ecolab	Peroxide compound
Perform	Schülke+	Peroxide compound

Reprocessing the Ardo Calypso breastpump



Additional information for Calypso

For the Calypso, check that there are no batteries in the compartment. If there are batteries, remove them and dispose of them according to standard guidelines.

Possible cleaners

Product name	Manufacturer / supplier	Ingredients
Surface disinfection / aldehyde-free		
Biguanid Fläche	Schumacher / Dengg	Quaternary compound
Incidin Plus	Ecolab	Alkylamine
TPH protect	Schülke+	Quaternary compound
Surface disinfection / aldehyde-containing		
Antiseptica Combi Surface	Antiseptics	Quaternary compound / aldehyde
Surface disinfection / ready-to-use / alcohol-resistant surfaces		
Incidin liquid	Ecolab	Alcohols
Mikrozyd AF liquid	Schülke+	Alcohols
Surface disinfection / ready-to-use / non-alcohol-resistant surfaces		
Acryl Des	Schülke+	Quaternary compound
Bacillol 30 Foam	Hartmann / Bode	Alcohols / alkylamine derivative
Surface disinfection / disinfectant wipes		
Kohrsolin extra tissues	Hartmann / Bode	Aldehyde / quaternary compound
Mikrobac Tissues	Hartmann / Bode	Quaternary compound
Sanicloth active wipes	Ecolab	Quaternary compound
Surface disinfection / oxidising agent		
Descogen liquid	Antiseptics	Peroxide compound
Incidin Active	Ecolab	Peroxide compound
Perform	Schülke+	Peroxide compound