CARUM

HOSPITAL-GRADE BREASTPUMP TO MEET THE HIGHEST DEMANDS







CARUM BREASTPUMP FOR HOSPITALS AND HIRE

The professional-use Carum breastpump was designed by our in-house IBCLC-led team to meet the most stringent requirements of mothers and lactation professionals. The result is a powerful and reliable pump with unique sensitivity.

Unique "Sensitive Programme"

With dual-piston technology, the Carum is unbeatable in the professionaluse breastpump market. Powerful and comfortable, with the unique "Sensitive Programme" for painful, inflamed nipples. Switch smoothly between stimulation and expression modes with independent adjustments at any time. Large color display with intuitive menu control and quiet operation for all pumping environments. The "Vacuum Seal" technology guarantees high safety and hygiene. No Carum pump parts need replacing between mothers. Saves costs, makes cleaning easier and ensures safety of multiple users.









Closed

System



- Fully automatic expression at the touch of a button
- Suction strength (vacuum) and frequency (cycle) settings can be adjusted individually and independently from one another
- "Sensitive Programme" in case of painful, inflamed nipples
- Uniquely gentle switching between stimulation and expression mode (the vacuum only increases by 30 mbar)
- It is possible to change between expression and stimulation mode at any time
- Pleasantly quiet during use
- Closed System

Thanks to the different background colours, it is easy to differentiate between the stimulation and expression phases on the 3.5-inch colour screen. The vacuum and cycle settings are clearly recognisable, as they are presented using bars. At night, or in darkened rooms, the dimming function makes it possible to read the display and express in comfort.

Particularly easy to handle and clean

- Easy to clean thanks to the smooth surface
- Large colour screen provides visual guidance
- Dimming function for comfortable expression at night
- Timing / stopwatch
- Also available with an optional mobile stand





CARUM BATTERY

BREASTPUMP WITH RECHARGABLE BATTERY

Franziska Wicki, Lactation Consultant IBCLC, Cantonal hospital Lucerne, Switzerland, has tested the Carum Battery and states: "I find the Carum Battery particularly practical and easy to use for mothers attending outpatient clinics. The pump can also be quickly set up for immobile mothers in the postnatal department. This lightens our workload significantly."



Lightens the workload of professionals


It is functionally identical to the existing high-quality Carum and also offers the same valuable features. A powerful integrated battery provides up to 8 hours of operation without charging. Freelance professionals can help the mother to express efficiently at home (in cases of engorgement, for instance), without the need for a mains connection. The Carum Battery therefore offers maximum power combined with excellent mobility.









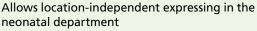

- Mothers can express where they can see their premature or sick child. This promotes better milk flow.
- Increased safety: it is not possible to pull out the wrong power cord

Lightens the workload of professionals


- 8 hours of battery life without recharging (24 x 20 min.)
- Looking for a power socket becomes unnecessary, e.g. in the patient's room at night
- The battery charging time is only 4 hours
- The status of the integrated battery is shown on the screen

Individual Phases


Ardo breastpumps can be adapted to the mother's personal needs and to the baby's individual suction action at any time. The vacuum intensity and cycle settings can be individually adjusted.

For mothers who want to decide for themselves

Mothers prefer being able to adjust the number of cycles and vacuum intensity themselves to suit their individual needs. Carum is the first breastpump to allow the cycle and vacuum intensity settings to be controlled independently from one another in both stimulation and expression mode.

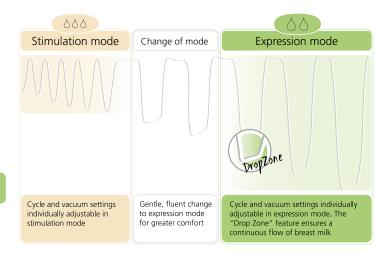

Sensitive Programme

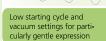

Simply unique: the "Sensitive Programme". Low and gentle starting cycle and vacuum settings especially for sore and tender nipples.





"Vacuum Seal" technology



High standards of hygiene and safety are achieved through "Vacuum Seal" technology

The proven "Vacuum Seal" technology serves as a 100 % barrier against bacteria and ensures that no breast milk or impurities can enter the connecting tube or pump. This provides reliable protection against contamination of the breast milk and thus prevents infection. The efficacy of the bi-directional leak-proof membrane pot was verified by an independent Swiss Laboratory in 2011.



Technical Data CARUM AND CARUM BATTERY

	Stimulation mode	Expression mode	Sensitive Programme
Vacuum (intensity of suction)	30 - 150 mbar Start level 60 mbar	30 - 330 mbar Start level 30 mbar higher than in stimulation mode	Start level 30 mbar
Cycles (frequency of suction)	72 - 120 per minute Start level 90 per minute	30 - 60 per minute Start level 45 per minute	Start level 30 per minute
	Carum	Carum Battery	Compliance / norms ISO 13485: 2003 CE (MDD 93/42/EEC) EMC tested (EN 60601-1-2) Ardo is WHO code compliant CE 0123
Art. no.	63.00.71	63.00.75	
Contents	Breastpump, European mains cable, protective carry case	Breastpump with battery, European mains cable, protective carry case	
Battery	-	Built-in (lithium)	
Measurements of pump (LxBxH)	302 x 240 x 180 mm	302 x 240 x 180 mm	
Power cable	Length 3 m	Length 3 m	
Power supply voltage / frequency	100 - 230 VAC / 50 - 60 HZ	100 - 230 VAC / 50 - 60 HZ	
Guarantee	3 years	3 years	

Practical accessories CONTAINER

TRANSPORT AND STORAGE OF CARUM

- Safe and hygienic
- Easy to clean
- With address field for drop shipment



EASY FREEZE HOLDER

FOR EXPRESSING INTO BREAST MILK BAGS

- Alternative to conventional bottle holders
- Suitable for Easy Freeze and small bottles
- Easy to clean by wiping with disinfectant



MOBILE STAND

FOR CARUM

- Stable and versatile
- Integrated storage for power cord
- Antistatic castors





Distributed in USA by: Comit Developers 116 Toledo Drive Lafayette, LA 70506, USA T 844-411-ARDO (2736) customer-service@ardo-usa.com www.ardo-usa.com Ardo medical AG Gewerbestrasse 19 6314 Unterägeri Switzerland T +41 41 754 70 70 F +41 41 754 70 71 www.ardomedical.com

