CARUM

Instructions for use | Instrucciones de uso | Mode d'emploi

en

Instructions for use Carum

3–15

es

Instrucciones de uso Carum

16–28


fr

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29–41

Explanation

- a** Carum Breastpump
- b** Connection opening for Pumpset
- c** Control panel with LCD
- d** Connection for power cord
- e** Bottle holder
- f** Easy Freeze holder (breast milk storage bag holder)
- g** Power cord
- h** Carum container
- i** Address field
- j** Pumpset (not included in the scope of delivery)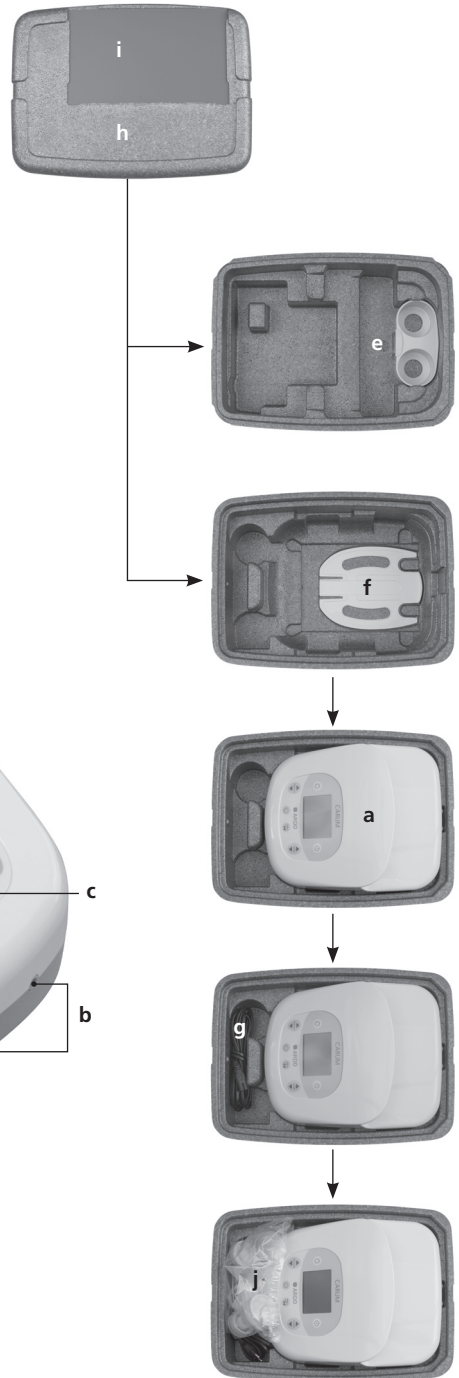

Thank you for choosing the gentle Carum breastpump from Ardo.

In these instructions you will find important and useful details for the application of the Carum breastpump.

When using electrical devices it is very important to observe the following safety regulations. Please take the time to carefully read the user manual before using the Carum for the first time.

CONTENT

1. Important Safety Information	5	6. Accessories	12
2. Product Description	6	7. Error Analysis / Error Display / Service Indicator	12
2.1. Suckling curve	6	7.1. Further possible errors when using the unit	12
3. Intended Use	7	8. Maintenance / Care / Cleaning	12
3.1. Indications	7	8.1. Visual inspection before switch-on	13
3.2. Contraindications	7	8.2. Cleaning/disinfection	13
4. Start-up / Operation	7	8.2.1. Checklist: Maintenance inspections for rentals	13
4.1. Icons	7	8.2.2. Checklist: Maintenance inspection for hospitals	13
4.2. Installation	8	9. Guarantee	13
4.2.1. Electrical connection	8	10. Technical Information	14
4.2.2. Connecting the Pumpset	8	10.1. Technical data	14
4.2.3. Installation of the bottle holder	8	10.2. Compliance	15
4.2.4. Installation of the Easy Freeze holder	8	10.3. EMC Test Report	15
4.3. Operation	9	11. Transport / Storage / Disposal	15
4.3.1. Switch on/switch off	9	11.1. Transport and storage conditions	15
4.3.2. Regulation of the vacuum and display	9	11.2. Service life and disposal	15
4.3.3. Cycle regulation and display	9		
4.3.4. Switch stimulation mode/expression mode	10		
4.3.5. "Sensitive Programme"	10		
4.3.6. Time button	10		
4.3.7. Backlight button	10		
4.3.8. Carum Battery	10		
5. Useful Tips	11		
5.1. Adjusting the vacuum (intensity of suction)	11		
5.2. Regulate the cycle	11		
5.3. Encouraging the let down reflex	11		
5.4. Pump efficiently	11		
5.5. Increase the quantity of milk	11		

1. Important Safety Information

Read all instructions before using this product!

Please keep these Instructions for use handy for reference at all times!

Carum is an electrically-operated breastpump for use in hospitals and for rentals. In the interest of your health and safety, and in order to avoid damage to the device, the following safety rules must be observed.

Safety Regulations

DANGER: To reduce the risk of electrocution

- Use only the original power cord from Ardo.
- Never use a damaged power cord.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not reach for a product if it has fallen into water. Immediately unplug from electrical outlet.
- Never operate Carum if it has dropped into water. In this case bring or send it to the nearest authorized service facility or the manufacturer for inspection.
- Do not use while in the bath or while taking a shower.
- Never open the Carum; danger of electric shock!

WARNING: To avoid damage, electric shock, injury or fire

- The unit is designed for indoor use only. Do not use outdoors.
- Never use Carum if it is not working correctly or if it has been dropped or damaged.
- Close supervision is necessary when this product is used near children or persons with disabilities; electrical devices are not toys.
- Plug the power cable into the Carum first and then into the mains socket.
- Do not leave product unattended when plugged into an outlet.
- After use, disconnect Carum from the outlet.
- Disconnection from the outlet is only guaranteed if the outlet plug is pulled out of the socket.
- Do not position the Carum so that it is difficult to operate the disconnection device.
- Keep the Carum, power cord and all accessories away from sources of heat.
- Carum should only be used for the purposes detailed in Section Intended Use.
- Only use a genuine Ardo Pumpset and genuine Ardo parts.
- Never drop or insert any object into any opening or tubing.
- Do not use Carum in an area where oxygen gas is in use or sprays are being applied.
- The device must not be modified without the express permission of the manufacturer.

WARNING: To avoid health risk and risk of injury.

- Inspect the breastpump and all appropriate pump components before each use.
- Do not shorten tubing.
- Never use Carum while sleeping or when dozy.
- If expressing is uncomfortable or painful, stop the expression and ask your lactation consultant for advice.
- Never use while pregnant, as pumping can induce labor.
- For cleaning, follow the instructions in Chapter Cleaning.

Battery option and car battery cable

- Carum or Carum Battery are not intended for outdoor use when using the car battery cable.
- Only use the electrically fused original Ardo car battery cable.
- Caution: Ensure that the car battery delivers 12 volts DC.
- In order to prevent an accident with serious injury, do not drive a vehicle while expressing, not even as a sidecar passenger.
- After use or after charging disconnect the device from the vehicle power system.


2. Product Description

- The Carum pump system consists of the breastpump and a Pumpset (single or double pumpset).
- The Carum breastpump is a quiet, user-friendly double piston pump for hospital use and for rental use.
- The unit is equipped with a robust plastic housing (Quality: VO) and a simplified, logically organised keyboard with a LCD display readout.
- The energy-saving electrical equipment is designed for outlet operation, for use with the car charger, or for (optional) battery use.
- The sophisticated controls allow each mother to enjoy a comfortable and efficient pumping experience (plus a tested suckling curve) thanks to the flexible system for adjusting vacuum and cycles.
- To emulate a baby's natural sucking behavior, the Carum allows pumping in 2 phases: stimulation mode and expression mode. Stimulation mode activates the milk flow, and the subsequent expression mode allows efficient pumping.
- Pumping with the "Sensitive Programme" is particularly suitable for sore nipples.
- The silicone pot integrated into the Pumpset ("Vacuum Seal" technology) ensures that no breast milk or air can get into the tube or Carum unit. The closed system is so secure that it constitutes a barrier against cross contamination (infection).


2.1. Suckling curve

The suckling curve is characteristically stable over the entire vacuum and cycle area and corresponds closely to your baby's suckling behaviour. Therefore the breast tissue is not additionally stressed by pumping.

Stimulation mode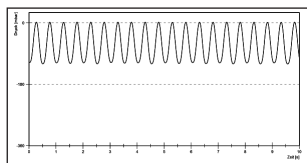

Expression mode

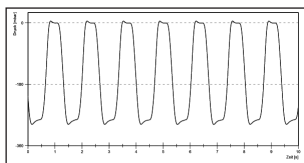3. Intended Use

3.1. Indications

The Carum breastpump is intended to be used by lactating women to express and collect milk from their breast. It can be used as a single pump and as a double pump. The unit is intended for indoor use only and is intended for multiple users.

3.2. Contraindications

None known: when the unit is used for the correct purpose.

4. Start-up / Operation

4.1. Icons

Follow instructions for use

Stand-by button

Adjust the vacuum

Adjust the cycle

Mode button (Switch to stimulation mode / expression mode / "Sensitive Programme")

Backlight button

Time button

Expression mode

Stimulation mode

Unit with the applied part BF

Compliance with general guidelines: MDD 93 / 42 EWG and MDD 2009

Manufacturer

The unit is compliant with the stipulations of the WEEE 2002 / 96 / EG guidelines

Power cord

Alternating current

Device of protection category II

IP21

IP 21 drip proof

Li-Ion

Contains a lithium ion battery


Device is recyclable

4.2. Installation

4.2.1. Electrical connection



- Outlet operation: Insert power cord (A) into the Carum (B) input socket and connect the local power plug to the wall outlet.
- Battery operation: Before using the battery for the first time, connect the unit to an outlet for 6 hours.
- The Carum can be used at maximum vacuum and maximum cycles for eight hours (24 x 20 minutes).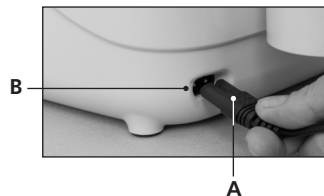

4.2.2. Connecting the Pumpset

Use of the Pumpset is detailed in the separate user manual: Ardo Pumpset.

- To use the single Pumpset:
Attach the tube connector (C) to the unit (D).
- To use the double Pumpset:
Attach the tube connector to the unit.
- The second opening of the tube connector (E) remains closed – by single and double pumping.

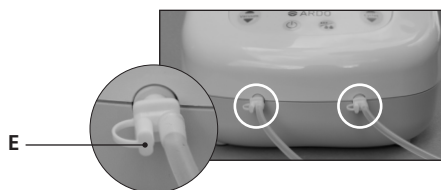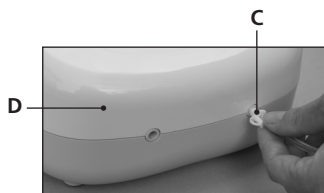4.2.3. Installation of the bottle holder

The bottle holder can be placed so as to be freestanding, or attached to the rear of the Carum.

4.2.4. Installation of the Easy Freeze holder


The Easy Freeze holder is used for suspending the pumpsets with breast milk storage bag or bottle. It is inserted at the rear of the Carum instead of the bottle holder.

4.3. Operation

Vacuum and cycles can be adjusted independently of each other. Select the adjustment which is the most comfortable for you.

- a Stand-by button
- b Adjust the vacuum
- c Vacuum information
- d Backlight button
- e Backlight information
- f Time display
- g Time symbol
- h Time button
- i Cycle information
- j Adjust the cycle
- k Mode button
- l Mode information / Expression
- m Mode information / Stimulation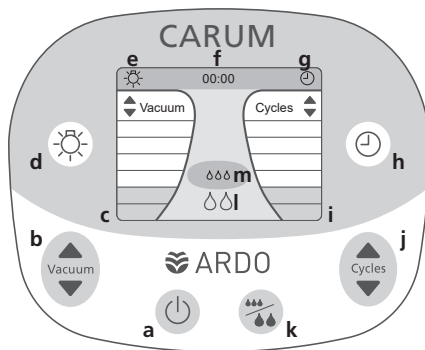

4.3.1. Switch on/switch off

Switch on the unit by pressing the stand-by button once.


A welcome page will be displayed.

When it is switched on, the breastpump is in stimulation mode (orange display). Medium vacuum and cycle level. To turn the unit off, press the stand-by button once again.

The Stand-by function is the same for battery operation, electrical outlet power supply or 12 volts DC power supply (car battery operation).

Display mode:


The operation mode is shown on the illuminated LCD display. The display turns off when the unit is switched off.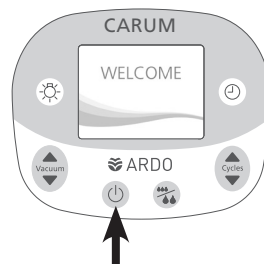

4.3.2. Regulation of the vacuum and display

The vacuum (suction strength) can be adjusted, **one level at a time**, by **repeatedly pressing**, or by **holding down** the vacuum button (to the left side of the key pad). The vacuum strength can be individually regulated in both stimulation mode and expression mode.


Pressing the up arrow increases the vacuum, pressing the down arrow reduces the vacuum. The vacuum strength is displayed on the bar chart to the left.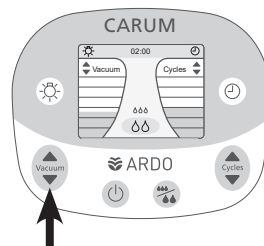

4.3.3. Cycle regulation and display

The cycles (frequency) can be adjusted, **one level at a time**, by **repeatedly pressing**, or by **holding down** the cycle button (to the right side of the key pad). The cycle can be individually regulated in both stimulation mode and expression mode.

Pressing the up arrow increases the cycles level, pressing the down arrow reduces the cycles level. The cycles level is displayed on the bar chart to the right.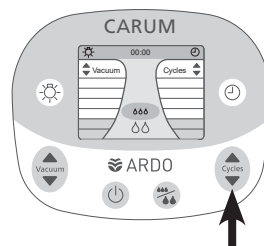

4.3.4. Switch stimulation mode/ expression mode

To change the stimulation mode display screen (orange display) to the expression mode display screen (green display), press the mode button just to the right of the stand-by button.

The change can be made at any time by pressing the mode button.


If the Carum is in stimulation mode and the mode button is not pressed, the unit will automatically change to expression mode after 2 minutes. Press the mode button to return to stimulation mode.

When the mode changes, the background colour of the LCD screen changes accordingly. In addition, the symbol relevant to stimulation or expression mode will be highlighted on the LCD display.

Background colour in stimulation mode is orange.


Background colour in expression mode is green.

In each separate mode of operation, vacuum strength and cycles level can be individually adjusted.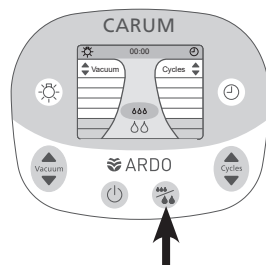

4.3.5. "Sensitive Programme"

The "Sensitive Programme" pumps with a barely noticeable vacuum and deep cycle; thus, expression with the "Sensitive Programme" is particularly suitable for sore nipples. Here vacuum and cycles can be gently increased.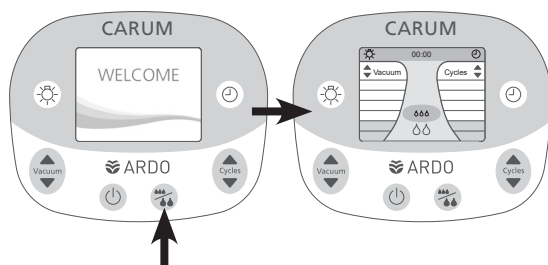


Procedure:

1. Press stand-by button.
2. Press the mode button while the welcome screen appears.

4.3.6. Time button


Timing starts automatically when the Carum is switched on. To stop timing, press the time button. Timing will remain de-activated until re-started by pressing the time button again, or until the unit is turned off. The timer will be reset to 00:00 when the time button is pressed for a second time.

Press the time button again to restart the timer. The timer can be switched on and off in either stimulation or expression mode.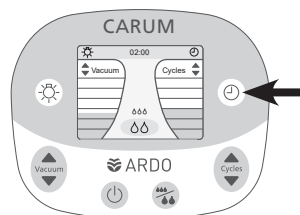


4.3.7. Backlight button

Using the Carum in a darkened room can be made more comfortable by dimming the intensity of the LCD. To reduce the intensity of the backlighting, press the backlight button (to the left of the LDC) once. The dim function can be turned on and off as required.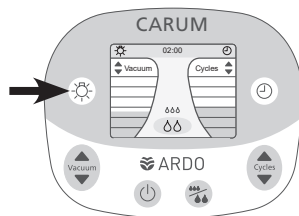

4.3.8. Carum Battery

The battery level is shown on the display. The battery charging time to the power supply is approximately four hours. The pump can be used while the battery is being charged. This will extend the charging time. The battery can be charged at any time without reducing the lifetime of the battery.

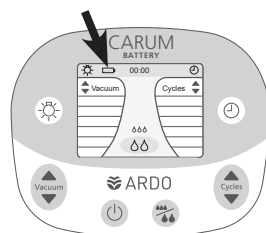5. Useful Tips

5.1. Adjusting the vacuum (intensity of suction)

Expressing milk should never be painful.

- Set the vacuum intensity at the highest level which is comfortable to you. If you are experiencing any discomfort or pain while pumping, you should decrease the vacuum intensity.
- Take care that the nipple is centred in the breast shell.
- Check whether the size of the breast shell fits your nipple. (See the User Manual: Ardo Pumpset).
- In the event that the milk is not released, try massaging the breast – this can be really effective.

Regulate the vacuum by touching the arrow keys.

- Intensify the vacuum, until you feel a very slight discomfort, and then reduce the intensity of the vacuum until pumping is comfortable.
- In the event of sore nipples or painful breasts, skip the stimulation mode. The Carum will then start in expression mode with an especially mild, gentle and low level vacuum.
- It is recommended that, in pump- or stimulation mode, you adjust the suction strength each time to suit your personal comfort level. This is because the vacuum level generated is sensitive to prevailing atmospheric pressure.

5.2. Regulate the cycle

- When babies start to breast feed, they suckle with rapid sucking motions and low suction – this causes the milk to release by stimulating the milk ejection reflex.
For this reason, the Carum will automatically start in stimulation mode, i.e. with a higher cycles level and a lower vacuum strength, mimicking the baby's natural behaviour.
- Select stimulation mode, choose the level of vacuum which feels most comfortable.
- When the milk begins to flow, the baby's sucking pattern changes. The baby sucks more slowly, but more strongly. Change to, and strengthen the vacuum and raise the cycles level by pressing the arrow until a comfortable pumping experience is achieved.
- If the milk flow falters, return to stimulation mode to prompt renewed milk flow.

5.3. Encouraging the let down reflex

The milk ejection reflex may occur later when pumping than when breast feeding. The following measures may be helpful.

- Have a warm drink.
- Relax and think about your baby.
- Look at a picture of your baby.
- Apply a warmed compress to the breast before expressing (e.g. Ardo Temperature Pack).
- Warm the cone or funnel of the breast shell before applying.

5.4. Pump efficiently

- Massaging the breast while using the breastpump can boost milk production, and promote better milk flow.

5.5. Increase the quantity of milk

- Double pumping
- Massage your breast while pumping.
- For the first 10 minutes, or until no more milk is flowing, use a double pump i.e. express from both breasts at once. Afterwards, switch to a single pump. Express from each breast for five minutes while massaging gently.

6. Accessories

Power cord Euro	Art. No. 63.00.130
Power cord UK	Art. No. 63.00.131
Power cord UL	Art. No. 63.00.132
Trolley - optional	Art. No. 63.00.129
Bottle holder	Art. No. 63.00.127
Easy Freeze holder	Art. No. 63.00.128
Various pumpsets	www.ardo-usa.com

7. Error Analysis / Error Display / Service Indicator

Identifiable errors or service/maintenance schedules are represented on the LCD by a warning triangle.

 Error	<p>Switch the unit off, and then on again. If the error message comes up again, return the unit to the supplier. The unit must be repaired by trained service personnel, or by the manufacturer.</p>
	<p>Error: the unit does not work on battery power i.e. the pump does not operate. Solution: Connect the unit to an outlet to recharge the battery. The battery charging time is ca. 4 hrs. Advice: The unit may be used while the battery is being charged, but this will extend the charging time.</p>

7.1. Further possible errors when using the unit

- Weak or absent vacuum
 Disconnect the Pumpset from the unit. Partly close the connector opening with a finger to test the suction. If suction is detectable, then the pump is working. Test the Pumpset in accordance with the separate instructions (only use the original Ardo Pumpset)
- The pump is not working/no data on the LCD display
 Check the power supply; if the unit still does not work, it is defective

Should any of these errors occur, and be confirmed, return the unit to the supplier immediately. The unit must be repaired by trained service personnel or by the manufacturer.

8. Maintenance / Care / Cleaning

The silicone pot integrated into the Pumpset constitutes a total barrier, and ensures that neither breast milk nor pathogenic agents can get into the pump unit. The unit is maintenance-free; care of the unit, and its accessories, is limited to inspection and cleaning.

The Carum breastpump can be repaired, but only by trained service personnel or by the manufacturer. For this reason, only the bottle holder, the Easy Freeze holder, the connector cable, and the car charging cable, are available for order as spare parts. See Section: 6. Accessories.

8.1. Visual inspection before switch-on

- Is the pump housing intact?
- Is the power unit undamaged?
- Is the accessory clean, complete and intact?
- Is the accessory correctly attached?

Ignoring these checks, or the cleaning instructions, may lead to risk of infection, electric shock or damage to the unit.

8.2. Cleaning / disinfection

- Carum breastpump:
Important: unplug the unit from the power supply before cleaning.
- The Carum is not waterproof. It conforms to enclosure IP (Ingress Protection) rating IP21.
Cleaning at home: As needed, wipe with a damp cloth, commercial detergent may be used.
Cleaning in the hospital: Disinfect by wiping with a non-phenol-based disinfectant.
Do not put the unit into the dish-washer or into a washing-up bowl. It is not possible to clean the pump or the power unit. Attempting to do so may cause the unit to malfunction or to become contaminated – requiring suitable disposal. (See section 11: Transport/Storage/Disposal).
- Accessory Pumpset: Please read the separate User Manual, Section: Cleaning.

8.2.1. Checklist: Maintenance inspections for rentals

Visual inspection of the pump for cleanliness	After every renting
Visual inspection of the pump for completeness	After every renting
Visual inspection of the pump for damage	After every renting
Simple operational checks of the pump (Pump runs properly, battery charged)	After every renting
Check of the vacuum strength – to be carried out by trained service personnel	Minimum every 2 years

8.2.2. Checklist: Maintenance inspection for hospitals

Visual inspection of the pump for cleanliness	Before every use
Visual inspection of the pump for damage	After every use
Check of the vacuum strength – to be carried out by trained service personnel	Minimum every 2 years

9. Guarantee

The warranty period of the Carum breastpump is 3 years, and the warranty period for the rechargeable battery 12 months from the date of purchase (the invoice with date is considered as warranty certificate).

General terms and conditions

This unit is covered by the Ardo medical AG warranty in respect of materials and manufacturing defects. The guarantee starts from the date of purchase. Defective materials will be replaced free-of-charge, during the life of the Guarantee, provided that the Guarantee has not been voided by unauthorised use or access to the unit. To ensure guarantee cover, and the flawless functioning of the unit, the instructions in the User Manual must be followed and only Ardo medical AG accessories used. The guarantee is void if the unit has been opened.

10. Technical Information

10.1. Technical data	
Pump dimensions L x W x H	320 x 240 x 180 mm
Pump weight	3.0 kg
Pump weight with battery	3.7 kg
Power cable length	3 m
Input voltage / Frequency	100 – 240 VAC \pm 10 % / 50 / 60 Hz
Power consumption	135 VA
Fuse input P / N	T 2.5 A / 250 VAC
Connector car adapter	12 VDC
Operating conditions	5–40 °C (41–104 °F) 10–93 % RH non-condensing 700–1060 hPa
Protection level (the pumpset is an applied part)	BF
Double insulated protection class	Class II
Classification in accordance with MDD 93/42/EEC	Class IIa
Mobile stand dimensions L x W x H	425 x 425 x 720 mm
Ground clearance	105 mm
Castors diameter	75 mm
Mobile stand weight	7.0 kg

	Vaccum (\pm 5 % ¹⁾)	Cycle (\pm 3)
Stimulation mode		
Start value	60 mbar 6 kPa 45 mmHg	90 / Min
Setting value	30–150 mbar 3–15 kPa 22–113 mmHg	72–120 / Min
Expression mode		
Start value	*	45 / Min
Setting value	30–330 mbar 3–33 kPa 22–248 mmHg	30–60 / Min
"Sensitive Programme"		
Start value	30 mbar 3 kPa 22 mmHg	30 / Min
Setting value	30–330 mbar 3–33 kPa 22–248 mmHg	30–60 / Min

* Expression mode uses the set point most recently selected by the user in stimulation mode and increases this vacuum by 30 mbar / 3 kPa / 22 mmHg

¹⁾ The vacuum level generated is sensitive to prevailing atmospheric pressure. The higher the prevailing atmospheric pressure, the more intense the generated vacuum – and vice versa.

10.2. Compliance

In accordance with Medical Devices Directive MDD 93/42 EEC, EN 60601-1, EN 60601-1-2, EN ISO 10079-1, EN 60601-1-6, EN60601-1-11, 2011/65/EU (RoHS)

10.3. EMC Test Report

See Page 42

CE
0123

MEDICAL – GENERAL MEDICAL EQUIPMENT AS TO ELECTRICAL SHOCK, FIRE AND MECHANICAL HAZARDS ONLY IN ACCORDANCE WITH ANSI/AAMI E560601-1 (2005, C1:09, A2:10), CAN/CSA C22.2 No.60601.1 (2008, R:2011)

11. Transport / Storage / Disposal

For the transport and storage of your Carum, we recommend using the original packaging. We further recommend to protect the products from UV (e.g. sunlight /halogen lamps), heat and dust.

11.1. Transport and storage conditions

Carum and Carum Battery

Temperature	-20 °C to +70 °C -4 °F to 158 °F
Relative humidity	10–93 % (non-condensing)
Atmospheric pressure	700–1060 hPa

11.2. Service life and disposal

The service life of the unit under normal conditions of use is 10 years, excluding the wear and tear parts. This device comprises electronic components and a rechargeable lithium ion battery. At the end of the service life of the device, the latter and its accessories must be disposed of according to the locally applicable regulations.

Do not throw used batteries into the garbage, but return them to the point of sale, or dispose of them according to the locally applicable regulations.

The device is subject to the requirements of the WEEE Directive.

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CE
0123

E358823

MEDICAL – GENERAL MEDICAL EQUIPMENT
AS TO ELECTRICAL SHOCK, FIRE AND
MECHANICAL HAZARDS ONLY
IN ACCORDANCE WITH
ANSI/AAMI E560601-1 (2005, C1:09, A2:10),
CAN/CSA C22.2 No.60601.1 (2008, R:2011)

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

TEST REPORT**IEC / EN 60601-1-2****Part 1-2: Electromagnetic Compatibility Requirements and Tests**

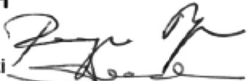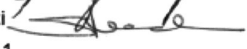Report reference No:	E1541-05-11	
Tested by:	B. Belegu	
Approved by management:	Dr. F. Stucki	
Date of issue:	26 April 2011	
Number of pages:	1 page	
Testing laboratory:	QUINEL AG	 S Schweizerischer Prüfstellendienst T Service Suisse d'essai S Servizio di prova in Svizzera S Swiss testing service STS 037
Address:	Grundstr. 2 CH-6343 Rotkreuz	
Testing location:	Rotkreuz Tel. 041-799 47 04	
Applicant's Name:	Ardo Medical AG, Mr. Christian Waser	
Address:	Gewerbstrasse 19, CH-6314 Unterägeri	
Manufacturer:	Ardo Medical AG	
Address:	Gewerbstrasse 19, CH-6314 Unterägeri	
Test specification:		
Standards:	IEC 60601-1-2:2007, EN 60601-1-2:2007	
Test procedure:	Type testing for Swiss and EU legal requirements	
Procedure deviation:	None	
Non-standard test method:	None	
Test-specification:	The used test setup fulfils the specification described in the relevant standards	
Test item description:	Breast Pump	
Trademark:	Ardo	
Model and/or type reference:	Carum	
Ratings:	100V-240V AC, 50Hz-60Hz	
Date of receipt of the test item(s):	06 April 2011	
Summary of testing:	Passed	
Applied standards:		
Title	Standard (up dated)	
RF disturbances	CISPR 22:2008, EN 55022:2010	
RF conducted disturbances	IEC 61000-4-6:2008, EN 61000-4-6:2009	
Radiated electromagnetic field	IEC61000-4-3:2006+A1:2007+A2:2010, EN 61000-4-3:2006+A1:2008+A2:2010	
Power frequency magnetic field	IEC 61000-4-8:2009, EN 61000-4-8:2010	
Electrical fast transient burst	IEC 61000-4-4:2004+A1:2010, EN 61000-4-4:2004+A1:2010	
Electrostatic discharge	IEC 61000-4-2:2008, EN 61000-4-2:2009	
Surge	IEC 61000-4-5:2005, EN 61000-4-5:2006	
Voltage dips - interruptions	IEC 61000-4-11:2004, EN 61000-4-11:2004	
Harmonic current emissions	IEC 61000-3-2:2005+A1:2008+A2:2009, EN 61000-3-2:2006+A1:2009+A2:2009	
Voltage fluctuations and flicker	IEC 61000-3-3:2008, EN 61000-3-3:2008	

Acceptance criteria/operating conditions:

During and after the test the apparatus should show no damage or loss of information and should function according to specifications.

This Test Report is, together with the Test Report E1541-05a-11 and E1541-05b-11, valid for the following types: Carum 1, Carum 2

Distributed in Canada by:
McArthur Medical Sales Inc.
1846 5th Concession West
LOR 1X0 Rockton Ontario, CA
T 519-622-4030
www.mcarthurmedical.com
mmsi@mcarthurmedical.com

Distributed in USA by:
Comit Developers
116 Toledo Drive
Lafayette, LA 70506, USA
T 844-411-ARDO (2736)
www.ardo-usa.com
customer-service@ardo-usa.com

 **ARDO**

Ardo medical AG
Gewerbstrasse 19
6314 Unterägeri
Switzerland
T +41(0)41 754 70 70
F +41(0)41 754 70 71
info@ardo-usa.com
www.ardo-usa.com

CE 0123 Class IIa