# ardo alyssa


## electric breastpump

Instructions for use | Instrucciones de uso | Mode d'emploi


 ardo  
SWITZERLAND

# Ardo Alyssa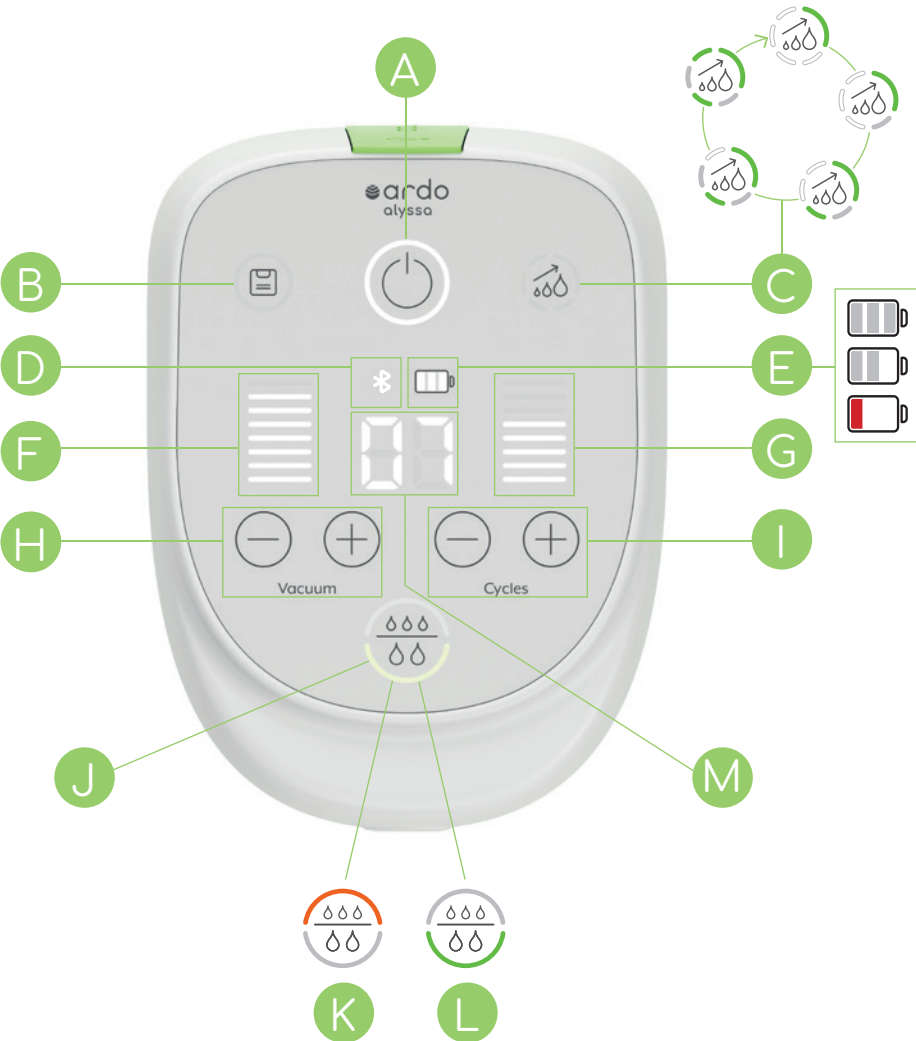

# Breast Shell Sizes

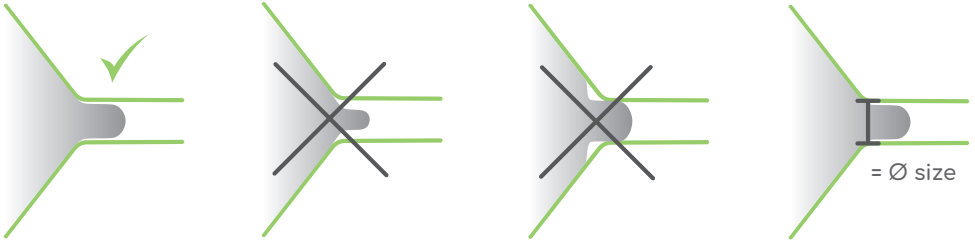en

The breast shell size is correct if:

- the nipple moves freely in the funnel and can follow the rhythmic movements of the breastpump.
- no part, or very little, of the areola is in the funnel.
- the milk flows and the breast feels uniformly soft after expression.

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## Table of Content

<b>1. Glossary</b>	<b>2</b>
<b>2. Indications for use</b>	<b>3</b>
<b>3. Product description</b>	<b>3</b>
3.1 Control elements and symbols	3
<b>4. Contraindications</b>	<b>4</b>
<b>5. Important safety information, warnings and precautions</b>	<b>4</b>
5.1 Information for the user	4
5.2 Warnings	4
5.3 Precautions	5
5.4 Cybersecurity	5
<b>6. Operation and expressing</b>	<b>5</b>
6.1 Start-up	5
6.2 Operation with a smartphone	5
6.3 Preparing the Ardo Alyssa breastpump	6
6.4 Operation and procedure for expressing	6
6.5 Memory function – Individual expressing at the touch of a button	7
6.6 Power Pumping	8
<b>7. Battery operation</b>	<b>8</b>
<b>8. Cleaning / maintenance</b>	<b>9</b>
<b>9. Milk storage guidelines</b>	<b>9</b>
<b>10. Useful tips for expressing</b>	<b>9</b>
10.1 Triggering the let-down reflex	9
10.2 Comfortable settings for vacuum	9
10.3 Combination of massage and expressing	10
10.4 Power Pumping	10
10.5 Pain when expressing	10
<b>11. Troubleshooting</b>	<b>10</b>
<b>12. Spare parts</b>	<b>11</b>
<b>13. Warranty</b>	<b>11</b>
13.1 General terms and conditions	11
<b>14. Technical information</b>	<b>11</b>
<b>15. Disposal</b>	<b>12</b>
<b>16. Trademark recognition</b>	<b>12</b>
<b>17. Electromagnetic compatibility</b>	<b>12</b>
<b>Emission and Immunity Tests</b>	<b>39</b>

# 1. Glossary

2-phase breastpump	Ardo Alyssa expresses in two phases: the stimulation and the expression modes, which imitate the sucking behaviour of the baby. This means a rapid cycle (suction frequency) with low vacuum (suction level) to stimulate the let-down reflex and a high vacuum with a slow cycle for efficient suction.
Stimulation mode	A rapid cycle (suction frequency) and low vacuum (suction level) start the breast milk flowing or trigger the let-down reflex.
Expression mode	Slower cycle (suction frequency) and high comfortable vacuum (suction level) for efficient expressing of breast milk once the let-down reflex has been triggered.
Cycles	Suction frequency / suction speed
Vacuum	Suction strength
Single pumping	Expressing from one breast
Double pumping	Expressing from both breasts
Let-down reflex	A reflex that starts the breast milk flowing.
Memory Plus	Saves the personal expression history including settings and time in stimulation and expression mode. When the breastpump is next switched on, the saved expression history starts.
Power Pumping	Interval pumping over one hour
'Vacuum Seal' technology	Closed pump system with hygiene barrier in the breast shell itself.
Contraindications	Conditions under which the device should not be used because the risk of use clearly outweighs any possible benefit
Warnings	Serious adverse reactions and potential safety hazards
Precautions	Information for the safe and effective use of the device

	Caution		Alternating current
	Consult instruction for use		Direct current
	Compliance with valid European regulations for medical devices		Temperature limit
	Manufacturer		Atmospheric pressure limitation
	Applied part BF type		Humidity limitation
	The device is subject to the provisions of the WEEE Directive 2002/96/EC		Keep away from sunlight
	Protection class		Keep dry
	Setting for the vacuum (suction level)		Setting for the cycles (suction frequency)
	Medical device		Device corresponds to the requirements (Part 15) of the Federal Communications Commission (FCC)
	MEDICAL — GENERAL MEDICAL EQUIPMENT AS TO ELECTRICAL SHOCK, FIRE AND MECHANICAL HAZARDS ONLY IN ACCORDANCE WITH ANSI/AAMI ES60601-1:2005 / ( R ) 2012 and A1:2012, C1:2009 ( R ) 2012 and A2:2010 ( R ) 2012, CSA CAN/CSA-C22.2 NO. 60601-1-14, IEC 60601-1-6:2010/AMD1:2013, CSA CAN/CSA-C22.2 NO. 60601-1-6:11 AMD 1, ANSI/AAMI HA60601-1-11:2015, CSA CAN/CSA-C22.2 NO. 60601-1-11:15		

## 2. Indications for use

The Ardo Alyssa is a powered breastpump to be used by lactating women to express and collect milk from their breasts. The Ardo Alyssa is intended to be used for a single user.

## 3. Product description

- a Ardo Alyssa breastpump
- b Ardo pumpset (included accessory)
- c Connection opening for pumpset
- d Power adapter with USB cable
- e Connection opening for USB power adapter
- f Bottle stand for pumpset

Ardo Alyssa is a vacuum pump to extract breast milk and is intended for personal use. It expresses in two phases (stimulation and expression modes) that imitate the sucking behaviour of the baby. With Ardo Alyssa you can express milk from one breast or both breasts at the same time.

The English text is the reference for the contents.

Instructions for the accessories: Only use Ardo pumpsets. Breast shells are available in the sizes  $\varnothing$  18 / 22 / 26 / 28 / 31 and 36 mm (see accompanying instructions for use for "Pumpsets and accessories"). Which breast shell sizes are included with delivery of your Alyssa is indicated on the packaging.

### 3.1 Control elements and symbols

The illustrations of the control elements and symbols are located on the fold-out page of the cover. The active elements or symbols light up.

<b>A</b>	On/Off button (standby button)	Switches the breastpump on and off
<b>B</b>	Memory Plus button	Tap the Memory Plus button: Saves the expression history of a pumping session. Memory Plus button lights up: Sequence is saved but is not displayed. Memory Plus button flashes: Saved expression history is displayed.
<b>C</b>	Power Pumping button	Automatically starts the Power Pumping program (interval pumping over an hour). The different segments light up green (during expressing) or white (during the pauses) depending on the progress of the interval pumping.
<b>D</b>	Bluetooth®	Shows the connection to the breastpump via Bluetooth®
<b>E</b>	Battery display	Shows the charge level of the rechargeable battery
<b>F</b>	Vacuum display	Step indicator for vacuum (suction level)
<b>G</b>	Cycle display	Step indicator for cycle (suction frequency)
<b>H</b>	Vacuum buttons	Increase $\oplus$ or decrease $\ominus$ the vacuum (suction level)
<b>I</b>	Cycle buttons	Increase $\oplus$ or decrease $\ominus$ the cycle (suction frequency)
<b>J</b>	Mode button	Displays the current mode and enables manual switching between stimulation and expression modes
<b>K</b>	Stimulation mode	Upper semicircle lights up orange in stimulation mode
<b>L</b>	Expression mode	Lower semicircle lights up green in expression mode
<b>M</b>	Timer	Displays the expressing time

## 4. Contraindications

None known

# 5. ⚠ Important safety information, warnings and precautions

Ardo Alyssa is an electrically operated, and reliable medical device. It has been developed and is manufactured according to CE guidelines and the relevant safety standards. For your safety and to prevent damage to the device and to avoid the risk of electric shock, injury or fire, please follow the safety instructions below.

### 5.1 Information for the user

Any complaints, problems, or safety concerns should be reported to the manufacturer or the distributor. According to Regulation (EU) 2017/745 on medical devices, any serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authorities of the Member State in which the user resides.

### 5.2 Warnings

#### Risk due to contamination / infection

- To prevent contamination of your breast milk, thoroughly wash your hands with soap and water before each use.
- Only use original Ardo pumpsets. If parts from other manufacturers are used, there is a risk of microbial contamination of parts as well as the breast milk because the parts are not compatible.
- If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
- Powered breastpumps that are designed for single users should never be rented or shared.
- The pumpset should not be shared.

#### Risk of health problems and serious injuries

- Do not use this device while driving a car (even as a passenger).
- Do not use this breastpump if you are overtired and may fall asleep during use, because this may lead to irritation of the nipples.
- Pregnant women should not use the breastpump, as pumping can induce contractions or premature labor.
- Make sure that this device is not used unsupervised by children or by people with reduced mental capabilities.

Ardo Alyssa may only be used for the purposes described in Section 2. For all uses apart from the intended use, all warranty and liability claims against the manufacturer are excluded.

#### Risk of an electric shock

- The Ardo Alyssa breastpump with the power adapter is only suitable for use indoors. Do not use outdoors.
- Only use this breastpump with the supplied Ardo original USB cable and power adapter.
- Ensure that the power adapter can always be easily accessed.
- In the case of the power adapter is loose or there is a gap, please stop using it immediately as there is a risk of electric shock
- Ardo Alyssa is not waterproof.
  - The device must therefore not be immersed in water or other liquids nor used or stored in locations where it could fall into water.
  - Do not use in the bath or when showering.
  - Do not clean in the dishwasher, under running water or by immersing in water.
  - The pump unit must not be used if it has fallen into water or is wet. If this happens, contact your service partner or the manufacturer!
- This breastpump must not be opened or modified in any way without the permission of the manufacturer. It does not contain any parts that are maintained by the user.
- Handle this device with care and avoid all shocks, blows or vibrations (e.g. do not allow it to drop onto the floor). If this device is nevertheless exposed to one of the mechanical effects described above, carefully inspect the device for damage before the next use. If the device is damaged, contact the point of sale or the customer service team at Ardo.

#### Risk of fire or toxic vapors

Keep the pump unit, USB cable with power adapter and accessories away from sources of heat.

**Risk of damage to the device or an unwanted change in the vacuum**

Do not insert or suck up any objects or liquids into the tube connector port of the device.

**Risk of injury due to the vacuum generated by the device or strangulation**

Make sure that the device is not used unsupervised by children or by people with reduced mental capabilities.

**Risk of incorrect mode of operation and reduction of the performance characteristics**

- Using this device near other devices or stacked with other devices should be avoided.
- Portable and mobile RF communication equipment (radio devices including their accessories such as antenna cables and external antennas) should not be used at a distance less than 30 cm (or 12 inches) from the parts and cords of the device labeled by the manufacturer.
- The use of accessories, power adapters and cords other than those defined or provided by the manufacturer of this device may result in increased electromagnetic noise emissions or a reduced electromagnetic immunity for the device and lead to a faulty mode of operation.

### 5.3 Precautions

**READ ALL INSTRUCTIONS BEFORE USING THIS BREASTPUMP.**

Not following these instructions for use may lead to damage to the device or personal injury.

**KEEP THESE INSTRUCTIONS FOR USE.**

For help with the initial setup or if something unexpected happens, please contact the point of sale or the customer service team at the manufacturer Ardo.

- If discomfort is felt, then suction can be broken by inserting a finger between the breast and the breast shell.
- If irritation or discomfort occurs, discontinue use and see a doctor.
- Before each use visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the parts have been replaced.
- Tubing – Inspect the tubing. If you find condensation in the air tube, wash and dry it immediately or replace the tubing. If you see breast milk inside the tubing or tube connector do not attempt to wash, clean or sterilize the tubing or tube connector. Instead contact the manufacturer.

### 5.4 Cybersecurity

The device is protected from piracy and unauthorized use. If you have any indication of a violation of cybersecurity protection please contact the manufacturer.

## 6. Operation and expressing

You can find more information about operating and using Ardo Alyssa at [www.ardo-usa.com](http://www.ardo-usa.com).

If you need additional help, please contact the point of sale, customer service representatives or a lactation consultant or midwife.

### 6.1 Start-up

**WARNING: Risk due to contamination / infection**

To prevent contamination of your breast milk, thoroughly wash your hands with soap and water before each use.

You can operate the Ardo Alyssa with either the USB cable that is included or the integrated rechargeable battery.

**Mains operation**

Push the flap **a** backwards, insert the USB cable **b** into the USB port on the breastpump and connect the power adapter to the mains power supply.

**Battery operation**

It is not necessary to fully charge the rechargeable battery before the first use. You can connect Ardo Alyssa with the USB power adapter and charge the rechargeable battery during the first five hours of use at the same time (see Section 7).

### 6.2 Operation with a smartphone

The MyArdo app allows you to operate the Ardo Alyssa breastpump. Configuration or special precautions are not required. The MyArdo app perfectly works along with other connected Bluetooth® devices, such as wireless headphones.

## Installing the MyArdo app

1. Download the MyArdo app free of charge from the App Store or Google Play on your mobile device and install it. MyArdo app is available for both iOS and Android operating systems. Please check in your App Store / Google Play if the version of your operating system is supported.
2. Open the MyArdo app on your mobile device and confirm the query about the instructions for use.

### Connect to the Ardo Alyssa breastpump

1. Ensure that Bluetooth® is activated on your mobile device.
2. Switch the breastpump on.
3. The serial number or code requested on the screen is located on the bottom of the Ardo Alyssa.
4. When the breastpump is connected with Bluetooth®, the corresponding symbol lights up .
5. Follow the instructions on the screen.

The Ardo Alyssa and the MyArdo app use modern technologies and implementation principles and do not store or transmit usage data to the cloud. Your data remains on your handset. The wireless interface uses encrypted communication for the control commands.

## 6.3 Preparing the Ardo Alyssa breastpump

### 6.3.1 Connecting the pumpset

#### Single pumping

Attach the tube connector to the Ardo Alyssa. The second opening of the tube connector stays closed.

#### Double pumping

Open the second opening of the tube connector and connect the tube of the second pumpset. Connect the tube connector to the Ardo Alyssa.

The use of the pumpset is described in the enclosed instructions for use for the "Pumpsets and accessories".

### 6.3.2 Visual inspection before each use

Every time you use the breastpump, check that ...

- ... the USB cable and the power adapter are undamaged.
- ... the pumpset is clean, correctly assembled and intact.
- ... the pumpset is correctly connected.

## 6.4 Operation and procedure for expressing

### Warning: Risk due to contamination / infection

To prevent contamination of your breast milk, thoroughly wash your hands with soap and water before each use.

### 6.4.1 Basic functions – step-by-step procedure

**1** Find a comfortable position for expressing, hold the pumpset on the breast shell and centre the nipple.

**2** Gently tap the On/Off button  for about 2 seconds to start expressing.

**3** The breastpump is in stimulation mode. This can be seen by the upper segment of the mode button that lights up orange.

**4** Adjust the vacuum (suction level) and the cycle (suction frequency) to what is comfortable for you. The vacuum and the cycle can be increased using the  button or decreased using the  button. By tapping the particular button, the intensity of the vacuum or cycle is changed by one step or one bar.

- 5 After two minutes, the stimulation mode  switches automatically to the expression mode .
  - a) If the breast milk starts to flow earlier, slowly tap the mode button  to manually switch to the expression mode.
  - b) If the let-down reflex could not be triggered after the preset two minutes, tap the mode button  to extend the stimulation phase by another two minutes.

---

- 6 The breastpump is in expression mode.  
This can be seen by the lower segment of the mode button that lights up green. 

---

- 7 Adjust the vacuum (suction level) and the cycle (suction frequency) to what is comfortable for you.  
The vacuum and the cycle can be increased using the  button or decreased using the  button. By slowly tapping the corresponding button, the intensity of the vacuum or cycle is changed by one step or one bar.  
Expressing is most successful if you select the vacuum and cycle settings that are comfortable for you.

---

- 8 The timer starts automatically when Ardo Alyssa is switched on and continuously shows the expressing time.   
Up to one minute the time is shown in seconds and from one minute it is shown in minutes.

---

- 9 Massage the breast while expressing. This improves optimal emptying of the breast during each pumping session.

---

- 10 Gently tap the On/Off button  for about 2 seconds to switch off the breastpump.  
The Ardo Alyssa shuts down automatically after 40 minutes.

## 6.4.2 Silent Start

If the Ardo Alyssa is switched on and is not connected to a pumpset, it runs very quietly but does not pump. This silent start enables you to connect to the MyArdo app without the Ardo Alyssa being active.

When the pumpset is inserted, you can start to express (see Section 6.4).

## 6.5 Memory function – Individual expressing at the touch of a button

You have found your ideal settings when your breast milk flows freely and you can express most successfully. Activating the memory button  saves your expression history. When you next switch the breastpump on, you can automatically express using the saved expression history in the stimulation mode and then in expression mode. The Ardo Alyssa breastpump stops automatically after the saved time.

You can nevertheless change the values for the vacuum and cycle manually at any time and define a new time for the length of the expression history. This takes you out of save mode. When you have finished expressing, you can save any changes to the expression history. If you do not save the new values, the previous settings are kept.

Whenever the Memory Plus button lights up, your expression history is saved, and when the breastpump is switched on, it automatically starts to pump with your individual data.

### Using the Memory Plus function – step-by-step procedure

- 1 **Expressing**  
Express using your favorite settings. 

---

- 2 **Saving an expression history**  
When you are finished expressing, tap on the Memory Plus button  to save your individual expression history. As confirmation, you will hear an acoustic signal, the button lights up and the breastpump automatically switches off.

---

- 3 **Accessing your expression history**  
Switch the breastpump on. You express directly with your saved data. The Memory Plus button  flashes slowly throughout.

### Deleting data

You can delete the saved data at any time if you do not want to express with the memory function. By slowly pressing the Memory Plus button  once, the saved data are deleted and the button no longer lights up.

## 6.6 Power Pumping


Automatic Power Pumping is a pre-defined one-hour interval pumping program – expressing three times with two pauses.

It starts by simply switching on the pump and pressing the Power Pumping button. The pump switches off automatically after the one-hour program. The program can be stopped prematurely by switching off the pump.

During the pauses you can remove the pumpset and move around freely.

The operation corresponds to the basic functions (see Section 6.4.1). The progress of the expressing is indicated by the green and white segments that light up.

Power Pumping imitates the frequency of breastfeeding / lactation of babies. There are times – usually in the evenings – when newborns or older babies want to be breastfed at 10-minute intervals (cluster feeding). More information can be found in Section 10.4.

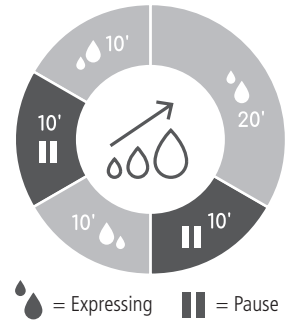### Power Pumping use – Step-by-step procedure

#### Start

Switch the breastpump on and press the Power Pumping button.

#### First expressing phase (20 minutes)

The first segment of the button starts to light up green and the breastpump starts in stimulation mode. You can adjust the vacuum and the cycle to what is comfortable for you. After two minutes, Ardo Alyssa switches to the expression mode. Again, you can select the vacuum and cycle that work for you. The color of the mode button  tells you which mode the breastpump is in.

#### First pause (10 minutes)

After 20 minutes a tone sounds and the first 10 minute pause starts. The second segment of the button lights up white. At the end of the pause, a reminder signal sounds.

You then have 5 minutes to activate the Power Pumping button and to continue expressing (during this time, the segments flash and a regular beep is emitted). Otherwise, the breastpump switches off.

#### Second expressing phase (10 minutes)

Ardo Alyssa expresses for 10 minutes. The settings for vacuum and cycle from the first expressing phase are used automatically. However, you can adjust these values at any time to suit your needs.

#### Second pause (10 minutes)

After 10 minutes a tone sounds and the second pause starts (see first pause for the sequence).

#### Third expressing phase (10 minutes)

The settings for vacuum and cycle from the second expressing phase are used. However, you can adjust these values at any time to suit your needs. After 10 minutes the Power Pumping stops and Ardo Alyssa switches off automatically.

The time indicator counts down the specified time in each phase.

## 7. Battery operation

- The breastpump has a built-in rechargeable battery. The battery cannot be replaced. The rechargeable battery is automatically charged when the supplied power adapter is connected to the mains power. The charging time is about 2 hours.
- The device can be operated while the battery is charging, in this case the charging time is extended. Only use the power adapter and the USB cable that are supplied with the breastpump.
- It is not necessary to fully charge the rechargeable battery before use.
- Air travel: The Ardo Alyssa has a lithium ion rechargeable battery. Please check if there are national restrictions on carrying lithium ion batteries when flying.

## Charging the rechargeable battery

Insert the USB power adapter into the breastpump and connect it to the mains power. When charging, the indicator moves from left to right.

3 bars (white): The battery is fully recharged. Expressing time (varies depending on the expressing settings):  
 - up to 8 times with a fully charged battery (15 minutes per session, vacuum level 6 and cycle level 6)  
 - up to 4 times with a fully charged battery (15 minutes per session, vacuum level 8 and cycle level 8)

2 bars (white): Expressing time (varies depending on the expressing settings):  
 - 2 to 4 times (15 minutes per session, vacuum level 6 and cycle level 6)  
 - 2 to 3 times (15 minutes per session, vacuum level 8 and cycle level 8)

1 bar (white): Expressing time (varies depending on the expressing settings):  
 - 1 to 2 times (15 minutes per session, vacuum level 6 and cycle level 6)  
 - 0 to 1 times (15 minutes per session, vacuum level 8 and cycle level 8)

1 bar (red): The breastpump must be recharged.  
 If the rechargeable battery is completely flat, the breastpump stops and switches off automatically.

## 8. Cleaning / maintenance

### Cleaning Ardo Alyssa

The pump body which houses the electrical units, which hold the motor and batteries, should be wiped down with a clean paper towel or soft cloth after each use. The pump body should never be put into water or other liquids for cleaning. It should also never be cleaned using a microwave sterilizer. Please follow the safety instructions in Section 5.

### Disinfection of Ardo Alyssa

Wipe the surfaces of the pump body with a non-phenolic, low-level disinfectant such as ethyl or isopropyl alcohol (70 – 90 %).

### Cleaning the pumpset

The cleaning instructions for the pumpset are described in the enclosed instructions for use for the "Pumpsets and accessories".

### Maintaining Ardo Alyssa

The breastpump does not have to be serviced.

## 9. Milk storage guidelines

Milk storage guidelines and recommendations are described in the enclosed instructions for use for the "Pumpsets and accessories".

## 10. Useful tips for expressing

### 10.1 Triggering the let-down reflex

The let-down reflex may occur later when expressing than when breastfeeding. The following measures may help the breast milk to flow in addition to the stimulation mode:

- Relax and think about your baby.
- Look at a video or photo of your baby.
- Massage the breast before expressing.
- Extend the stimulation mode (see Section 6.4.1).
- Place a warm compress on the breast before expressing.

### 10.2 Comfortable settings for vacuum

What determines the level of the vacuum is your personal well-being and not the maximum vacuum that can be set. Therefore, expressing using your comfortable vacuum setting.

### 10.3 Combination of massage and expressing

You may massage the breast(s) while using the breastpump to improve optimal emptying of the breast(s) during each pumping session.

### 10.4 Power Pumping

You may use the unique automatic Power Pumping function of the Ardo Alyssa over one hour (see Section 6.6) in addition to your usual breastfeeding or expressing routine.

- Express from both breasts at the same time if possible. Double pumping increases the milk volume.
- Use the special bra for double pumping or you may modify an old bra to accommodate the breast shells.
- Use the 10 minute pauses to relax, drink a little, eat something, have a shower, read a book, listen to some music, do small chores around the house and so on.
- You may use the Power Pumping program 1–2 times a day.
- The expressed breast milk should be stored according to the storage recommendations provided.

### 10.5 Pain when expressing

Expressing must not be painful. If it hurts, the following possible reasons must be eliminated:

- The vacuum is set too high. Adjust the settings so that they are comfortable for you (see Section 10.2).
- The nipples are not centered. Ensure that your nipples are in the middle of the breast shell.
- The funnel of the breast shell does not fit. It is too small or too big. The breast shell size is right if:
  - your nipples move freely in the funnel and follow the rhythmic movements of the breastpump (about 3 mm space between funnel and nipple).
  - none or only a little bit of the areola is located in the funnel.
  - the breast milk flows and the breast feels soft all over after the expressing.
- The breastpump is switched on **before** attaching the breast shell. As a result, the nipple is difficult to centre.

Order the correct size breast shell from Ardo, your specialist retailer or your qualified consultant. Ardo has the following breast shell sizes in stock: ø 18 / 22 / 26 / 28 / 31 and 36 mm (see cover).

## 11. Troubleshooting

### On the display the message "EE" appears (electronic error)

- Switch the breastpump off and on again.
- If the message appears again, contact your point of sale, your retailer or the customer service team at Ardo.

### The breastpump suction is too weak or absent

- Ensure that the pumpset has been correctly assembled and is not damaged (see instructions for use for the "Pumpsets and accessories").
- Check that the pumpset is correctly attached to the breastpump. When single pumping (using one pumpset), close the second opening of the tube connector.
- Check whether the USB cable and the power adapter are correctly attached and connected to the power supply.

### App connection delay or failure

- The Ardo Alyssa is visible for the app and ready to connect only when switched on or charging. If the MyArdo app does not find or connect to the Ardo Alyssa, check that the Bluetooth® is activated on the phone and that the smartphone supports Bluetooth® 4.0 or higher. Ensure the MyArdo app has the required Android or iOS privileges.
- If the MyArdo app loses connection a waiting circle is indicated and the MyArdo app will try to reconnect. At any time the device can be operated directly via device interface.

If these steps still do not help you, please contact your point of sale, your retailer or the customer service team at Ardo.

## 12. Spare parts

Ardo Alyssa breastpump (body)	Art. no. 63.00.450
USB cable with EU power adapter	Art. no. 63.00.541
USB cable with UK power adapter	Art. no. 63.00.542
USB cable with US power adapter	Art. no. 63.00.543
USB cable with AU power adapter	Art. no. 63.00.544
USB cable with CN power adapter	Art. no. 63.00.545
USB cable with KR power adapter	Art. no. 63.00.546
Pumpset	See instructions for use for the "Pumpsets and accessories"

## 13 Warranty

There are legally prescribed minimum warranty periods from the date of purchase that are specific for each country. The warranty coverage is limited to the first purchaser.

### 13.1 General terms and conditions

Ardo medical AG guarantees that their products are free from material and manufacturing defects. The warranty period starts from the date of purchase. Faulty materials are replaced free of charge during the warranty period, provided that the warranty has not been voided by improper use or interventions on the device. In order to safeguard the warranty coverage and ensure perfect functioning of the device, the instructions for use must be followed and only Ardo accessories may be used. The warranty is voided if the device is opened or manipulated in any way.

## 14 Technical information

### Pump unit

Vacuum pump type	Moderate vacuum, low flow rate
Vacuum range at 0–2000 MAMSL	30–330 mbar / 23–248 mmHg
Weight	320 g
Battery capacity	2600 mAh
Dimensions	128 × 94 × 67 mm
Protection class	Class II
Applied part	Class BF
Bluetooth® interface	IEEE 802.15.4 Bluetooth® class II wireless interface Radio frequency band: 2.4 GHz ISM Transmission power: +3 dBm Compatibility: Bluetooth® 4.0 and higher Range: in the same room within 10 meter
IP protection class	IP 22 (protected against solid objects of up to 12 mm and protected against falling water droplets if the housing is tilted at up to 15°)
Life time of device incl. power adapter	400 hours
Operating conditions	+41°F (+5°C) to +104°F (+40°C) 10% to 93% RH 700–1060 hPa
Storage and transport conditions	+14°F (-10°C) to +140°F (+60°C) 10% to 93% RH 700–1060 hPa

## Power adapter

Input voltage	100–240 V, 50/60 Hz, max. 0.5 A
Output voltage	5 V / 2.0 A (*2.1 A)
Max. cable length	1.5 m
IP protection class	IPX0 (no protection against solid objects or water)
Approved models	EU: MKC-0502000VEUU (#99.05.025) EU: XY-0030A (#99.05.025)* UK: MKC-0502000VUKU (#99.05.026) UK: XY-0030AUK (#99.05.026)* US: MKC-0502000VU (#99.05.027) US: XY-0030A (#99.05.027)* AU: MKC-0502000VAUU (#99.05.028) AU: XY-0030AAU (#99.05.028)* CN: MKC-0502000XXX (#99.05.029) CN: XY-0030ACN (#99.05.029)* KR: MKC-0502000XXX (#99.05.030) KR: XY-0030AKC (#99.05.030)*

## 15. Disposal

The device can be returned to the manufacturer or be dismantled and recycled in an environmentally responsible manner by a recycling company specialising in electric devices. There are no special or increased risk factors. Waste must be disposed of in compliance with local regulations.

## 16. Trademark recognition

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth® SIG, Inc., and any use of such marks by Ardo medical AG is under license.

Apple is a trademark of Apple Inc., registered in the U.S. and other countries. Android is a trademark of Google LLC.

## 17. Electromagnetic compatibility

The Ardo Alyssa breastpump meets all the relevant requirements for the applicable standards relating to interference emissions and immunity for operation in medical and domestic environments.

More information about electromagnetic tests is given in the section 'Emission and Immunity Tests' (page 39).

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# Emission and Immunity Tests

The Alyssa Pump is classified for following tests and test levels:

- CISPR 11 group 1 class B.
- IEC61000-3-2 Harmonic current emissions.
- IEC61000-3-3 Voltage changes, fluctuations and flicker compliant for the full range of supply voltages from 100V-240V at 50/60Hz.
- IEC61000-4-2 Electrostatic Discharge compliant for
  - $\pm 8$  kV contact discharge
  - $\pm 2$  kV,  $\pm 4$  kV,  $\pm 8$  kV,  $\pm 15$  kV air discharge
- IEC61000-4-3 radiated RF electromagnetic field compliant with
  - 10 V/m; 80 MHz – 2,7 GHz; 80 % AM at 1 modulation
- IEC61000-4-3 Immunity to proximity field from RF wireless communications equipment compliant
  - TETRA 400 385MHz @ 27V/m
  - GMRS/FRS460 450MHz @ 28V/m
  - LTE Band 13,17 710,745,780MHz @ 9V/m
  - GSM800/900, TETRA 800, iDEN 820, CDMA850 LTE Band 5 810,870,930MHz @ 28/7m
  - GSM1800, CDMA1900, GSM 1900, DECT; LTE Band 1, 3, 4, 25 UMTS 1720, 1845, 1970MHz @ 28V/m
  - Bluetooth®, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7 2450 MHz @ 28V/m
  - WLAN 802.11 a/n 5240, 550, 5785 MHz @ 9V/m
- IEC61000-4-4 Immunity to fast transient/burst on a.c. mains
  - $\pm 2$  kV 100 kHz repetition frequency
- IEC61000-4-4 Immunity to fast transient/burst on SIP/SOP
  - $\pm 1$  kV 100 kHz repetition frequency
- IEC61000-4-5 Immunity to Surge
  - line to line mains  $\pm 0,5$  kV,  $\pm 1$  kV
  - line to ground SIP/SOP  $\pm 2$  kV
  - line to ground mains  $\pm 0,5$  kV,  $\pm 1$  kV,  $\pm 2$  kV
- IEC61000-4-6 Immunity to conducted disturbances induced by RF fields on a.c. mains
  - 3 V; 0,15 MHz – 80 MHz; 6 V; ISM and amateur radio bands 0,15 MHz to 80 MHz
  - 80 % AM at 1 kHz
- IEC61000-4-6 Immunity to conducted disturbances induced by RF fields on SIP/SOPs.
  - 3 V; 0,15 MHz – 80 MHz; 6 V; ISM and amateur radio bands 0,15 MHz to 80 MHz 80 % AM modulation
- IEC61000-4-8 Immunity to power frequency magnetic fields
  - 30 A/m 50 or 60 Hz
- IEC61000-4-11 Voltage dips, short interruptions and voltage variations immunity
  - 0 % UT; 0,5 cycle @ 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°, compliant
  - 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: @ 0°, compliant
  - 0 % UT; 250/300 cycle, charging may be temporarily interrupted.

No test deviations to the basic standard. No special points to be observed for compliance with EMC.

Device is not tested against IEC60601-2-2 ANNEX BB Equipment used in the proximity of HF surgery devices. Not relevant for the application.

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

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T +41 41 754 70 70  
F +41 41 754 70 71  
info@ardo.ch  
www.ardomedical.com

Distributed in USA by:  
Ardo medical, Inc.  
13800 Coppermine Road  
1st Floor  
Herndon, VA 20171, USA

T 844-411-ARDO (2736)  
support@ardo-usa.com  
www.ardo-usa.com