


# Troubleshooting the Ardo Pumpset

It is important that the pumpset is assembled correctly for your Ardo pump to work properly. We have a video showing the assembly on our Facebook page (Ardo USA) and on YouTube (youtube.com/ardousa). Watch the video and ensure that you are assembling the pumpset correctly, then use this troubleshooting guide if you are still having an issue to determine which part(s) need to be replaced.

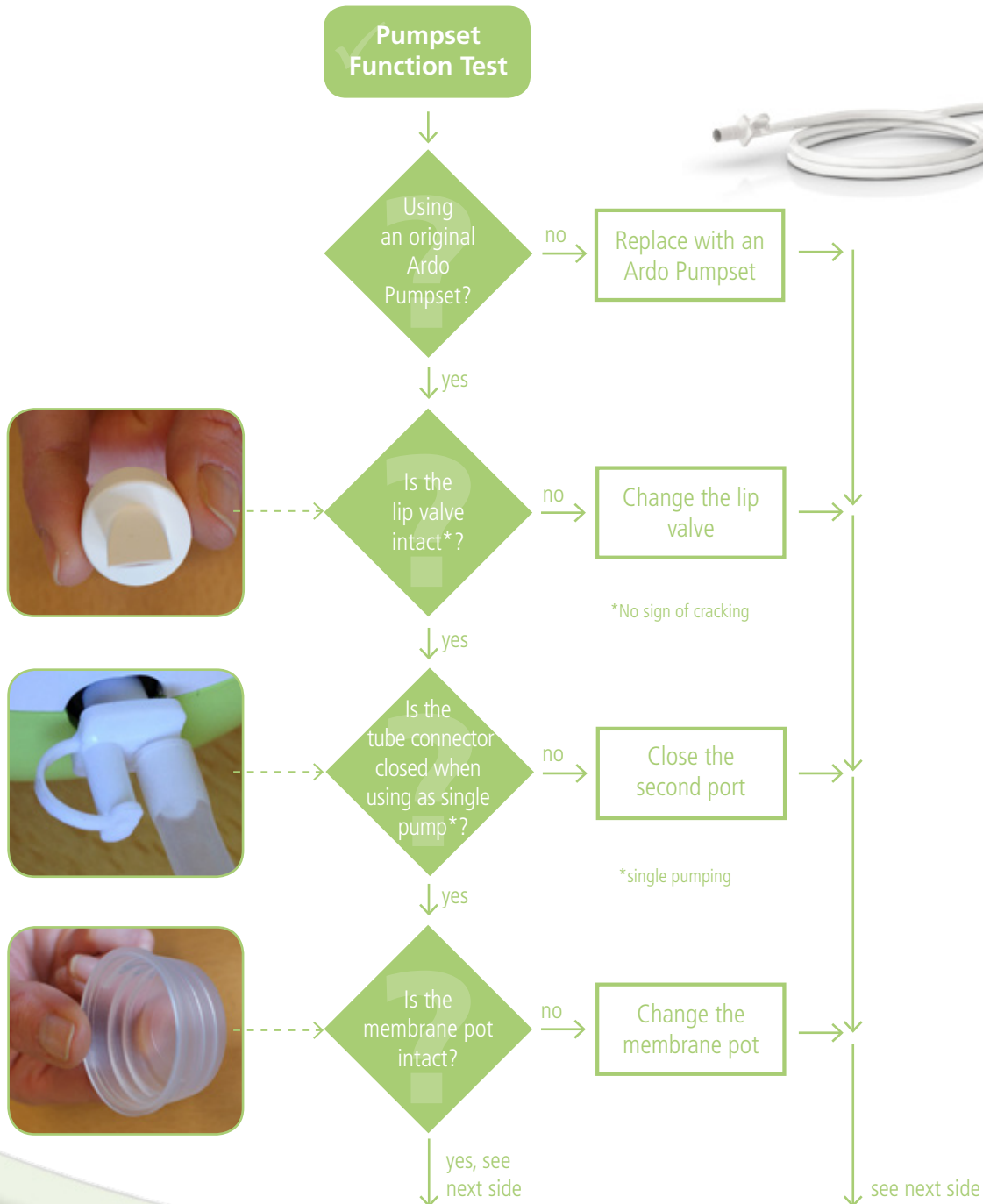



is the membrane pot assembled correctly\*?

no

Assemble with the smooth side facing outwards

\*With the smooth side facing outwards

yes



Is the membrane pot lying flat in the breast shell?

no

Insert the membrane pot flush into the breast shell

yes



Is the top cap fully closed?

no

Take top off, line up the mark on the top with the mark on the breast shell and turn to the right.

yes



Is the tube connector properly connected?

no

Push in the double port completely

yes

Switch on the unit



Does the membrane pot contract and expand?

no

Pumpset defective: change

yes

Thank you! Your Calypso is working properly